## Aptima<sup>®</sup> Urine Specimen Collection Kit

## HOLOGIC°

Collection procedure guide

## For male and female urine specimens

Patient should not urinate for at least 1 hour prior to specimen collection.



Direct patient to provide **first-catch** urine (~20 - 30 mL of initial urine stream) into urine collection cup. Collection of larger volumes of urine may result in specimen dilution that may reduce test sensitivity. Female patients should not cleanse labial area prior to providing specimen.





Remove cap and transfer 2 mL of urine into urine specimen transport tube using the pipette provided.





The correct volume of urine has been added when the fluid level is between black fill lines on urine specimen transport tube label.



Discard pipette. Tightly screw cap onto tube. This is now known as the "processed urine specimen."

Hologic provides this collection procedure guide as a general informational tool only; it is not an affirmative instruction or guarantee of performance. It is the sole responsibility of the clinician/ laboratory to read and understand the appropriate package insert and comply with applicable local, state and federal rules and regulations.

## hologic.com | diagnostic.solutions@hologic.com | 1.888.484.4747

DS-01697-001 Rev. 004 © 2020 Hologic, Inc. All rights reserved. Hologic, Aptima and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. This information is intended for medical professionals in the U.S. and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your local Hologic representative or write to diagnostic.solutions@hologic.com.